

CONTENTS

General.....	2
Information on User's manual	2
Symbols used in this User's Manual	3
Intended Use	3
Product Description	4
Fitting the Filter System.....	4
Cleaning the Coarse Filter	5
Changing the Fine Filter	6
Disposal.....	6
Disclaimer.....	7

GENERAL

INFORMATION ON USER'S MANUAL

Read this user's manual through carefully before using the filter system for the first time.

Follow the cleaning instructions in particular.

Keep the manual in a safe place close to the filter system so that you can refer to it immediately if necessary.

SYMBOLS USED IN THIS USER'S MANUAL

Important information is denoted by symbols in this user's manual. Be sure to follow these instructions to avoid material damage.

NOTICE

This symbol denotes situations that may lead to material damage or damage to the filter system.

IMPORTANT

This symbol denotes information, tips and instructions for the efficient, error-free use of the filter system.

INTENDED USE


The ***filtersystempoint*** is used to filter the therapy air of the point respiratory therapy device. The filter system is more effective than a standard filter because of its size and the double filtering of the respiratory therapy air. This filter provides relief for patients with allergies.

PRODUCT DESCRIPTION

The **filtersystempoint** (art. no. 0000 7847) consists of:

1. Filter cassette
2. Coarse filter (art. no. 0000 2108)
3. Fine filter (art. no. 0000 2109)

The fine filter has a filtration efficiency of 99.5 % with particles up to 1 µm.



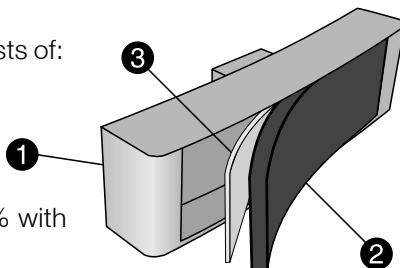


FITTING THE FILTER SYSTEM

1. Remove the standard filter.
2. Insert the filter cassette.

NOTICE

Ensure that the filter system is installed correctly. The cassette must be flush with the bottom of the device when fitted (3).

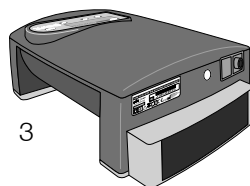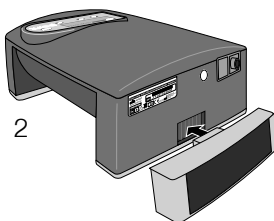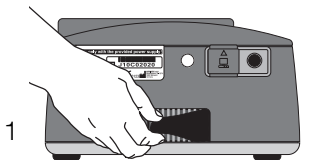

CLEANING THE COARSE FILTER

Clean the coarse filter once a week. To do this, proceed as follows:

1. Remove the filter cassette and take out the coarse filter from the filter cassette.
2. Clean the coarse filter with mild soapy water. Do not use any other agents!
3. Rinse the coarse filter thoroughly with clear water.
4. Let the coarse filter dry completely in the air.
5. Insert the coarse filter into the filter cassette.
6. Attach the filter cassette onto the device.

CHANGING THE FINE FILTER

The white, double layer fine filter cannot be cleaned and should be examined visually once a week. It must be changed if the display message →**FILTER**← appears or every 2 months or even more frequently if it is very dirty. To do this, proceed as follows:



1. Remove the filter cassette and take out the coarse filter from the filter cassette.
2. Remove the fine filter and replace it with a new one.
3. Insert the coarse filter into the filter cassette.
4. Attach the filter cassette onto the device.

IMPORTANT

Refer to the point user's manual for use to find out how to delete the →**FILTER**← display message.

DISPOSAL

The filter system and the packaging can be disposed of with the normal household waste.



DISCLAIMER

HOFFRICHTER GmbH is not liable for consequences in terms of safety, reliability and performance of the product where:

- interventions, modifications, extensions, calibration, repairs and maintenance are carried out by persons not authorized by us,
- other manufacturers' accessories and spare parts are used that have not been approved by us for use on the product,
- the product is used other than as described in the user's manual or
- the hygiene and cleaning instructions described in the user's manual have not been complied with.

Statutory guarantee rights remain unaffected by this.

HOFFRICHTER GmbH
Mettenheimer Strasse 12/14
19061 Schwerin
Germany
Phone: +49 385 39925-0
Fax: +49 385 39925-25
E-mail: info@hoffrichter.de
www.hoffrichter.de