

VEGA CPAP MACHINE

Quick Set Up



Plug in power cord and switch on mains power supply (Standby Mode)



Press once to start machine

How to Change Settings

Switch OFF machine using ON/OFF Button



Press once; 'off' will appear top right of display

Press at the same time: ON/OFF + RAMP + ARROW DOWN



Press once; 'Settings' will appear top left of display and the machine will start to operate

Use the MENU button to select desired setting:



Use to move between different settings

Use the UP and DOWN arrows to change values:



Use these buttons to change values

Settings

Pressure	3.5 - 20
Altitude	Refer to Instruction Manual
Fine Adjustment	Refer to Instruction Manual
Delay Time (RAMP)	10 mins to 60 mins in 10 min intervals
Ramp Start Pressure	3.5 – 20
Compliance Hrs	Refer to Instruction Manual
Operation Hrs	Refer to Instruction Manual
Unit Hrs	Refer to Instruction Manual

Use ON/OFF button to close the 'Settings' display



Press once; the machine will stop operating